

How to Use CrystLCare™ Regeneration+ Dental Strips

For Cavity Prevention & Dental Sensitivity Treatment



Dental Professional – Initial In Office Application

1 Clean & Prepare




Prepare/clean tooth/teeth requiring treatment.

2 Open & Remove



Ensure disposable gloves are **dry**. Open pouch by pulling apart upper sealed end of package. Remove strip immediately before use.

3 Moisten

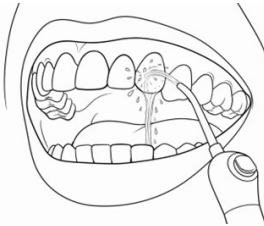


Rinse teeth with water.
(1 mL dissolves strip)

4 Apply




Apply either side of strip onto affected area. Fold excess behind or around teeth. Gently press until strip adheres. Lightly remoisten and reposition if necessary.

5 Wait

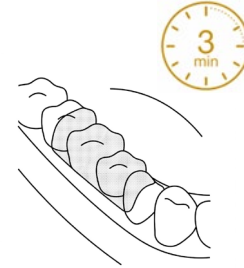

Leave in place for at least **3 min**. Instruct patient to expect gradual dissolution.

6 Protect

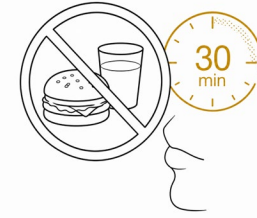

Excess material may be expectorated or suctioned. Instruct patient not to eat or drink for 30 minutes.

7 Instruct




Review with patient:

- Teeth/sites to treat at home
- How to place strip correctly
- How often to apply
- Duration of treatment
- When to call dental office


Patient - Home Use

1 Clean & Prepare

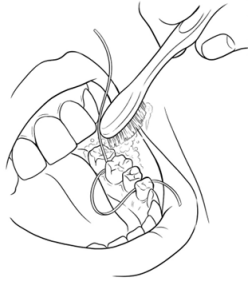


Brush teeth with toothpaste, rinse with water and floss before use.

2 Open & Remove

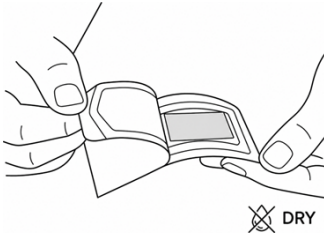

Dry hands prior to use. Open pouch by pulling apart upper sealed end of package. Remove strip immediately before use.

3 Moisten

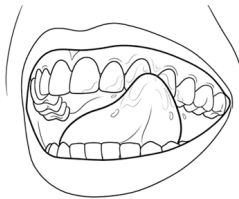


Moisten teeth with water.
(1 mL dissolves strip)

4 Apply



Apply either side of strip onto affected area. Fold excess behind or around teeth. Gently press until strip adheres. Lightly remoisten and reposition if necessary.

5 Wait

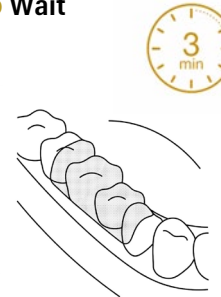

Leave in place for at least 3 min. Strips are dissolvable. Do not swallow strip whole.

6 Protect



Spit out any excess if desired. Do not eat or drink for 30 minutes.

7 Use as Directed



Follow your dental professional's directions for treatment frequency and duration.